Ovulation Test | Instruction for Use


[REF] EZW2-S

The Easy@Home Ovulation Test is a qualitative test used to predict when there is a LH surge, and in turn, when you are likely to ovulate.

For *in vitro* self-testing use only. For external use only.

HOW DOES IT WORK?

The LH (Luteinizing Hormone) in the urine of normal women will increase dramatically in the middle of the menstrual cycle. The LH increase triggers ovulation when the egg is released periodically from normal fertile women. The LH surge occurs about 24 to 48 hours before you actually ovulate. A positive LH test may predict the most fertile window time in your cycle. When the strip is immersed into a urine specimen, capillary action carries the specimen to migrate along the membrane. A colored line will appear at the Control Zone if the test has been performed properly. The Control Zone also works as a reference line for you to find the LH peak time when the test line is as dark or darker than the control line.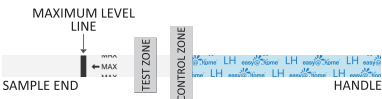

CONTENTS OF THE TEST KIT

- One pouch containing a test strip and a desiccant. The desiccant is for storage purposes only and is not used in the test procedures.
- Instruction for Use.

WHAT ELSE DO YOU NEED?


- A clean, dry, plastic or glass container to collect the urine.
- Timer (watch or clock)

WHEN TO START TESTING?

The length of the menstrual cycle is the duration from your first menstrual bleeding day to the day before the next bleeding begins. Determine the length of menstrual cycle before test. Count the first day of bleeding as Cycle Day 1. Please refer to the chart below to determine when you should start testing. If you do not know your cycle length, you may start testing since the Cycle Day 5 until the LH surge has been detected.

Cycle Chart

| Cycle Length | Day of Cycle to Begin Testing |
|--------------|-------------------------------|
| 21 days | day 5 |
| 22 days | day 6 |
| 23 days | day 7 |
| 24 days | day 8 |
| 25 days | day 9 |
| 26 days | day 10 |
| 27 days | day 11 |
| 28 days | day 12 |
| 29 days | day 13 |
| 30 days | day 14 |
| 31 days | day 15 |
| 32 days | day 16 |
| 33 days | day 17 |
| 34 days | day 18 |
| 35 days | day 19 |
| 36 days | day 20 |
| 37 days | day 21 |
| 38 days | day 22 |
| 39 days | day 23 |
| 40 days | day 24 |

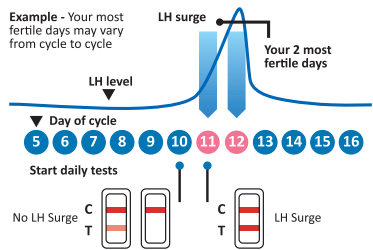Example: If your cycle is normally twenty-six days, the Cycle Chart indicates testing should begin on day 10.

The following calendar shows how to determine day 10.

Sample calendar

| | | | | | | |
|----|----|----------|----|------------|----|----|
| 1 | 2 | 3+ day 1 | 4 | 5 | 6 | 7 |
| 8 | 9 | 10 | 11 | 12# day 10 | 13 | 14 |
| 15 | 16 | 17 | 18 | 19 | 20 | 21 |
| 22 | 23 | 24 | 25 | 26 | 27 | 28 |

3+ = The first day of menstrual bleeding
12# = The day to begin testing (Day 10)

WHEN TO COLLECT URINE FOR THE TEST?


Any urine specimen is appropriate for ovulation testing.

HOW TO COLLECT THE URINE SAMPLE?

Urine may be collected in any clean, dry, plastic or glass container.

HOW TO PERFORM THE TEST?

- The test strip and urine have to be at room temperature prior to testing.
- Remove the test strip from the sealed pouch.
- Dip the white absorbent end into the urine sample up to the black 'Max' line until you see the dye begin to rise into the testing window (Est. time: 5-10 seconds).
- Lay the test strip flat on a clean, dry, and non-absorbent surface.
- Read the results in 5 minutes after immersion or after any excess dye has completely cleared the testing window. Do not read after 5 minutes.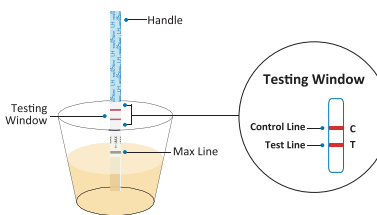

HOW TO READ THE RESULTS?

Positive (LH Surge):


If two color lines are visible, and the test line is as dark or darker than the control line, you will probably ovulate in the next 24-48 hours.

Negative (No LH Surge):

Only one color line appears in the control zone, or the test line appears but is lighter than the control line. This means there is no LH surge.

Invalid:

No visible lines at all, or there is a visible line only in the test zone but not in the control zone. Repeat with a new test kit. If test still fails, please contact the distributor or the store where you bought the product, with the lot number.

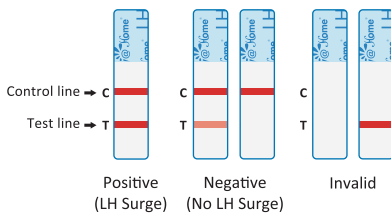STORAGE AND STABILITY

- Store at 4°C to 30°C (39.2°F - 86°F) in the sealed pouch up to the expiration date.
- Keep away from direct sunlight, moisture and heat.
- DO NOT FREEZE.
- The test should be used immediately after opening.

QUESTIONS & ANSWERS

- There is a big difference between the control lines of two different ovulation tests. Is this a concern?**
No. Variations in the color of the control line will not affect the test result. It is only effective to compare the color of the test line to that of the control line of the same test on the day the test is performed.
- Can test results be interpreted after more than five minutes?**
Test results must be read at 5 minutes or a few minutes more if excess red dye still needs to clear the test window. Though a positive result should not change for several days, negative result may change to a false positive after the end of the testing period, which would not be an accurate reading. If any question for the test result, please contact us via email: support@premom.com.
- How long should I continue to perform the test?**
Continue testing for at least 5 days or until the LH surge has been detected.
- A pink background color and vertical streaking appeared in the result area during the testing period. Is this a concern?**
No. Each urine sample will vary in its chemical makeup, as will the humidity of the air in testing chamber (room). Such variations in physical conditions can cause the vertical streaking and/or the pink-rose background color, but will not affect the test results. As long as the control line appears within five minutes, the test is working properly.

5 Do alcohol or common medications affect the test?

No, but you should consult your physician if you are taking any medication.

6 I am now using the basal body temperature method (BBT). Does this test replace BBT?

The elevated basal body temperature primarily indicates that ovulation has already occurred. This test indicates that ovulation is about to occur within 24 to 48 hours.

7 I have received a positive result and had intercourse during these fertile days. I have not become pregnant. What shall I do?

Many factors can affect your ability to become pregnant. Often you may need to use the test kit for 3-4 months. You and your partner should consult your physician if pregnancy is not achieved after 3-4 months.

8 Will the amount of liquid I drink affect the result?

Yes. We suggest that you limit your fluid intake for about two hours before you collect your urine. Heavy intake of fluids prior to testing will dilute the hormone in your urine.

9 Can I use first morning urine for testing?

Unlike pregnancy testing, the FMU (first morning urine) is NOT recommended to use for ovulation testing. Luteinizing hormone (LH) levels usually start to rise in the early morning and may not be easily detectable in the first morning urine. The ideal time to collect a urine sample is from 10 a.m. to 8 p.m., although testing can safely be performed at any time.

NOTE: It is suggested to test twice a day when it is close to the LH surge day so that you won't miss it; the LH surge can be very short for some women. If you choose to test twice a day, you may test once around noon and another time in the early evening. We recommend collecting the urine specimen at about the same time each day.

PRECAUTIONS

- This kit is for external use only. Do not swallow.
- Discard after first use. The test strip cannot be used more than once.
- Do not use test kit beyond the expiration date.
- Do not use the kit if the test pouch is punctured or not fully sealed.
- Keep out of the reach of children.
- Please follow local regulations to discard used tests.

LIMITATIONS

- The tests results are more accurate when the test procedures are precisely followed.
- This test may not be used as a form of birth control.
- Consult a doctor if irregular or unusually long cycles are experienced.
- This test is a qualitative screening for human urine. It is not designed to determine the quantitative concentration of LH.
- Contaminated specimen or test device or wrong operation or interfering substances in the urine may cause false result.
- Some drugs for infertility treatment, such as 17 β -Estradiol, Progesterone and Estradiol Valerate, may cause false result.

PERFORMANCE CHARACTERISTICS

ACCURACY

429 urine specimens were tested with Easy@Home Ovulation Test and a legally marketed test in a clinical Reference Laboratory. The correlation between Easy@Home Ovulation Test and marketed test was 99.8%.

| | | Marketed test | | |
|----------------|-------|---------------|-----|-------|
| | | + | - | Total |
| Easy@Home Test | + | 144 | 0 | 144 |
| | - | 1 | 284 | 285 |
| | Total | 145 | 284 | 429 |

Agreement of the positive with marketed test: 99.3 %(144/145)

Agreement of the negative with marketed test: 100%(284/284)

Total Agreement with marketed test: 99.8%(428/429)

SENSITIVITY AND SPECIFICITY

The Ovulation Test will detect hLH in human urine at the level higher than 25 mIU/mL. The test has been calibrated by W.H.O Standard (2nd WHO International Standard (IS) 80/552).

No cross-reaction appears when there is hFSH (200 mIU/mL) or hTSH(250 mIU/L) in specimen.

INTERFERING SUBSTANCE

The following potentially interfering substances were added to LH negative and positive specimens.

| | | | | | |
|----------------------|---------|---------------|---------|--------------|---------|
| Acetaminophen | 20mg/dL | Caffeine | 20mg/dL | Hemoglobin | 20mg/dL |
| Acetylsalicylic acid | 20mg/dL | Gentisic acid | 20mg/dL | Tetracycline | 20mg/dL |
| Ascorbic acid | 20mg/dL | Glucose | 2mg/dL | Ampicilline | 20mg/dL |
| Atropine | 20mg/dL | | | | |

None of the substances at the concentration tested interfered in the assay.

BIBLIOGRAPHY

- Sievert LL, Dubois CA. "Validating signals of ovulation: Do women who think they know, really know?" Department of Anthropology, Machmer Hall, UMass Amherst, Amherst, Massachusetts. Am J Hum Biol. 2005 Apr 22;17(3):310-320
- Ruiz Anguas J, Carballo Mondragon E, Anta Jaen E, Diaz Calvillo J, Kably Ambe A. "Usefulness of luteinizing hormone levels to determine the oocyte quality" Ginecol Obstet Mex. 2005 Jan; 73:28-35

MEANING OF SYMBOLS ON PACKAGE

| | | | | | | | |
|--|--------------------------|--|------------------------------|--|--|--|---|
| | Keep away from Sunlight | | Do not re-use | | Store between 4°C - 30°C (39.2°F - 86°F) | | Keep Dry |
| | Tests per kit | | Manufacturer | | Batch code | | Authorized representative in the European Community |
| | CE marking of conformity | | Consult instructions for use | | In vitro diagnostic medical device | | Expiry date |
| | Catalog number | | Date of manufacture | | | | |

DISTRIBUTED BY

Easy Healthcare Holdco India Private Limited
Suite 302, 3rd Floor Trinity Complex, 5AC, 512
5th A Cross, 2nd Block HRBR Layout,
KalyanNagar, Bengaluru, Karnataka 560043

QUESTIONS OR COMMENTS?

Email: support@premom.com
To learn more, please visit us at

www.premom.in

EC REP Qarad BV
Cipalstraat 3
2440 Geel, Belgium

Guangzhou Wondfo Biotech Co., Ltd.
No.8 Lizhishan Road, Science City,
Luogang District, 510663,
Guangzhou, P.R.China

Importer: BIO STATE INC.
Address of importer: Shop No-16, G.F, Arjun Arcade Phase-1, Shopping Complex, IP Colony, Sector-30-33 Faridabad, Haryana (India) - 121003
Importer's Phone Number: 0129-4054435
Email: regulatory@biostate.in
Import licence No.: IMP/IVD/2022/000392

Scan the code to get the manual in more languages

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