Pregnancy Test | Instruction for Use

REF EZW1-S

Easy@Home Pregnancy Test is a rapid pregnancy test, which you can easily carry out yourself. It detects the presence of human chorionic gonadotropin ("hCG"), which appears in urine very early during pregnancy. For *in vitro* self-testing use only. For external use only.

HOW DOES IT WORK?

Human chorionic gonadotropin (hCG) is a hormone produced by the developing placenta shortly after conception and secreted into the urine. The pregnancy test contains antibodies which specifically react with this hormone.

When the strip is immersed into a urine specimen, capillary action carries the specimen to migrate along the membrane. When hCG in the sample reaches the Test Zone of the membrane, it will form a Rosy Pink line. Absence of this Test Line suggests a negative result.

To serve as a procedure control, a colored line will appear at the Control Zone if the test has been performed properly.

CONTENTS OF THE TEST KIT

- One pouch containing a test strip and a desiccant. The desiccant is for storage purposes only and is not used in the test procedures.
- Instruction for Use.

WHAT ELSE DO YOU NEED?

- A clean, dry, plastic or glass container to collect the urine.
- Timer (watch, clock or smartphone).

WHEN TO COLLECT URINE FOR THE TEST?


Any urine specimen is appropriate for pregnancy testing. However, the first morning urine specimen is preferred because of its highest concentration of HCG.

HOW TO COLLECT THE URINE SAMPLE?

Urine may be collected in any clean, dry, plastic or glass container.

HOW TO PERFORM THE TEST?

- The test strip and urine have to be at room temperature (15-30°C or 59°F - 86°F) for testing.
- Remove the test strip from the sealed pouch.
- Dip the white absorbent tip of the test into urine up to the black "MAX" line and keep the tip in urine until dye rises into the results window. (Est. time: 5-10 seconds)
- Lay the test strip flat on a clean, dry, and non-absorbent surface.
- Read the results between 3 and 5 minutes after immersion.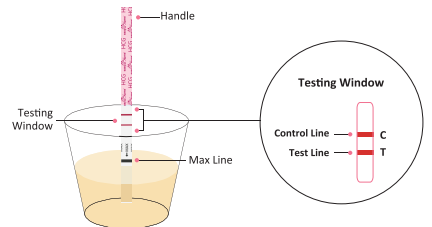

Positive results maybe observed in as soon as 1 minute. Do not read results after 5 minutes.

HOW TO READ THE RESULTS?

Positive (pregnant) :


Distinct color lines appear in the Control and Test Zones. It indicates that you are pregnant. The color intensity of the test lines may vary since different stages of pregnancy have different concentrations of hCG hormone.

Negative (not pregnant) :

Only one color line appears in the Control Zone and no Rosy Pink line on the Test Zone. This indicates that no pregnancy has been detected.

Invalid:

No visible line at all, or there is a visible line only in the Test Zone and not in the Control Zone. Repeat with a new test kit. If the test still fails, please contact us for assistance.

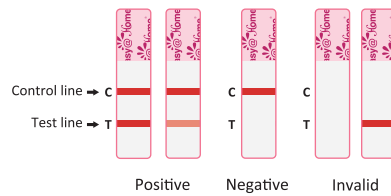NOTE:

If the color in the Test Zone is very faint, it is recommended to repeat the test two days later.

WHAT TO DO WHEN YOU KNOW THE RESULT?

IF THE RESULT IS POSITIVE (TWO LINES)

Consult your physician to confirm your pregnancy after evaluating all clinical and laboratory findings. In certain conditions, the test may give a false positive result, and your physician will also give you further information and advise you on your pregnancy. See also "Limitations"

IF THE RESULT IS NEGATIVE (ONE LINE)

If you have reasons to believe that you are pregnant (e.g. your period is late), you can repeat the test a few days later. If the test is still negative, consult your physician. See also "Limitations".

LIMITATIONS

- Like any other diagnostic procedure, a confirmed pregnancy diagnosis should only be made by a physician after evaluating all clinical and laboratory findings.
- If a urine sample is too dilute (e.g. low specific gravity) it may not contain hCG up to the cutoff level. If pregnancy is still suspected, another test should be repeated two days later.
- Low concentration of hCG in a very early pregnancy can give a negative result. In this case, another test should be repeated two days later.
- Elevated levels of hCG can be caused by a few conditions other than pregnancy. Therefore, the presence of hCG in urine specimen should not be used alone to diagnose pregnancy, unless these conditions have been ruled out.
- A normal pregnancy cannot be distinguished from an ectopic pregnancy based on hCG levels alone. Also, spontaneous miscarriage may cause confusion in interpreting test results.
- Sterility treatments, based on hCG, may cause false results. Consult your physician on how to do test properly if you are taking any hormone medication.
- This test is a qualitative screening for human urine. It is not designed to determine the quantitative concentration of HCG.
- Contaminated specimen or test device or interfering substances, or wrong operation in the urine may cause false result.

STORAGE AND STABILITY

- Store at 4°C to 30°C (39.2°F - 86°F) in the sealed pouch up to the expiration date.
- Keep away from direct sunlight, moisture and heat.
- DO NOT FREEZE.
- The test should be used immediately after opening.

QUESTIONS & ANSWERS

- There is much difference between the control lines of two tests. Is this a concern?**
No. Variations in the color of the control line will not affect the test result.
- Can test results be interpreted after more than five minutes?**
No. Test results must be read between 3 and 5 minutes. Though a positive result should not change for several days, a negative result may change to a false positive within minutes after the end of the testing period, which would not be an accurate reading. It is always best to read the results between 3 and 5 minutes and then discard the test to avoid confusion.
- A pink background color and vertical streaking appeared in the result area during the testing period. Is this a concern?**
No. Each urine sample will vary in its chemical makeup, as will the humidity of the air in the testing chamber (room). Such variations in physical conditions can cause the vertical streaking and/or the pink-rose background color but will not affect the test results. As long as the control line appears within 5 minutes, the test is working properly.

PRECAUTIONS

- This kit is for external use only. Do not swallow.
- Discard after first use. The test strip cannot be used more than once.
- Do not use the test kit beyond the expiration date.
- Do not use the kit if the test pouch is punctured or not fully sealed.
- Keep out of the reach of children.
- Please follow local regulations to discard used tests.

EXPECTED RESULTS

Negative results are expected in healthy non-pregnant women and healthy men. Although the hCG levels in normal early pregnant women are variable, Easy@home Pregnancy Test is capable of detecting pregnancy as early as 1 day after the first missed menses.

PERFORMANCE CHARACTERISTICS

Sensitivity

Easy@Home Pregnancy Test will detect urine hCG concentration of 10mIU/mL or greater.

Specificity

The specificity of the Easy@Home Pregnancy Test was determined from cross-reactivity studies with known amounts of Luteinizing Hormone (hLH), Follicle Stimulating Hormone (hFSH), and Thyroid Stimulating Hormone (hTSH). Negative results were obtained from all tests conducted with 300mIU/ml hLH, 300 mIU/ml hFSH and 1000 µIU/ml hTSH.

BIBLIOGRAPHY

- Canfield RE, Morgan FJ, Kammerman S, Bell JJ, Agosto GM. Studies of human chorionic gonadotropin. Recent Prog Horm Res. 1971; 27:121-164
- Ross GT. Clinical relevance of research on the structure of human chorionic gonadotropin. American Journal of Obstetrics and Gynecology 1977; 129:795.

MEANING OF SYMBOLS ON PACKAGE

	Keep away from Sunlight		Do not re-use		Store between 4°C - 30°C (39.2°F - 86°F)		Keep Dry
	Tests per kit		Manufacturer		Batch code		Authorized representative in the European Community
	CE marking of conformity		Consult instructions for use		In vitro diagnostic medical device		Expiry date
	Catalog number		Date of manufacture				

DISTRIBUTED BY

Easy Healthcare Holdco India Private Limited
Suite 302, 3rd Floor Trinity Complex, 5AC, 512
5th A Cross, 2nd Block HRBR Layout,
KalyanNagar, Bengaluru, Karnataka 560043

QUESTIONS OR COMMENTS?

Email: support@premom.com
To learn more, please visit us at

www.premom.in

EC REP Qarad BV
Cipalstraat 3
2440 Geel, Belgium

Guangzhou Wondfo Biotech Co., Ltd.
No.8 Lizhishan Road, Science City,
Luogang District, 510663,
Guangzhou, P.R.China

Importer: BIO STATE INC.
Address of importer: Shop No-16, G.F, Arjun Arcade Phase-1, Shopping Complex, IP Colony, Sector-30-33 Faridabad, Haryana (India) - 121003
Importer's Phone Number: 0129-4054435
Email: regulatory@biostate.in
Import licence No.: IMP/IVD/2022/000392

Scan the code to get the manual in more languages

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