# Pee Cup

## User Manual

REF PUC-50

English

### HOW DOES IT WORK?

Premom Pee Cup is a sample collection tool designed to make urine collection easier for women.


### PREPARATIONS

- Ensure you have a flat surface nearby to place your cup on after collecting the sample.
- Wash hands thoroughly before collecting urine sample.
- Wash and dry the Pee Cup completely to remove all residue.

### HOW TO USE

1. Fully expand the bottom of the Pee Cup. Wash it with warm water and mild detergent before first use.
2. Thoroughly dry the cup and hold it by the handle.



3. Position the pee cup to collect your urine until it is about half full. Remove the cup and finish urinating into the toilet.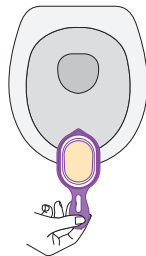

### WARNINGS AND PRECAUTIONS

- Use the Pee Cup only for its intended purpose of collecting urine samples. Do not use it for eating, drinking, or any other purpose.
- Do not share the Pee Cup for safety and hygienic reasons.
- Do not use the Pee Cup if it arrives damaged or becomes damaged in any way.
- Keep the Pee Cup out of reach of children; it is not a toy.
- Pee can naturally stain or discolor the Pee Cup over time.
- DO NOT place the Pee Cup in dishwasher.

### PRODUCT COMPONENT

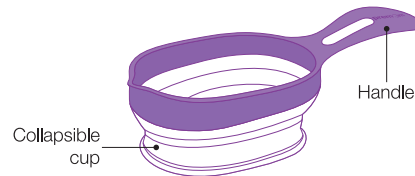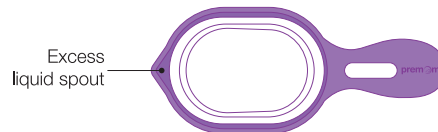### CONTENTS

- Pee Cup(s)
- User Manual

### MEANING OF SYMBOLS ON PACKAGE

Keep away from sunlight

Manufacturer

Store between 4°C - 30°C (39°F - 86°F)

Catalog#

Date of manufacture

Keep dry

Lot number

Recyclable

Consult instruction for use

### QUESTIONS OR COMMENTS?

Distributed by: **Easy Healthcare Holdco India Private Limited**  
Suite 302, 3rd Floor Trinity Complex, 5AC, 512 5th A Cross, 2nd Block HRBR Layout, KalyanNagar, Bengaluru, Karnataka 560043

Email: [support@premom.com](mailto:support@premom.com)  
To learn more, please visit us at

[www.premom.in](http://www.premom.in)

Made in China

Scan the code to get the manual in more languages

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Huiyilong Mold Co., Ltd